




Extended Dwell Peripheral IV Catheter  
Bedside Sign

leaderflex

Date: \_\_\_\_\_

Site: \_\_\_\_\_

Size: \_\_\_\_\_

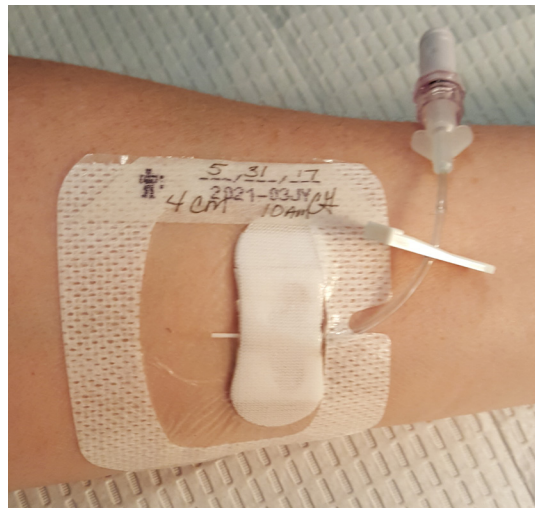

- Up to 29 dwell days in the absence of signs and symptoms of infection, infiltration and extravasation.
- Do not use device for power injection.
- Dressing change needed every 7 days or as needed. Keep dressing dry and intact. If dressing becomes loose, soiled or wet, it must be changed.<sup>1</sup>
- Do not take blood pressure in arm with Leaderflex.
- Do not start peripheral IVs below Leaderflex.
- Flush per organizational policy.<sup>1</sup>
- Do not use for continuous vesicants or irritant infusions. Use for infusions that are ideal for peripheral devices.

Your Vygon Representative: \_\_\_\_\_

<sup>1</sup> Infusion Nurses Society. Infusion Therapy Standards of Practice. 2016.

